

TSIII $\varnothing 5.5$ Implant

Bigger than $\varnothing 5.5$, smaller than $\varnothing 6.0$

OSSTEM[®]
IMPLANT


Specification of $\varnothing 5.5$ Fixture




System	TSIII
Diameter	$\varnothing 5.5$
Length	6, 7, 8.5, 10, 11.5, 13, 15mm
Surface	SA / CA
Mount	Pre / No

Indication of $\varnothing 5.5$ Fixture


1. To overcome Poor Initial Fixation of $\varnothing 5.0$ Fixture : Replace 5.0 Fixture with 5.5 Fixture when the initial fixation is weak

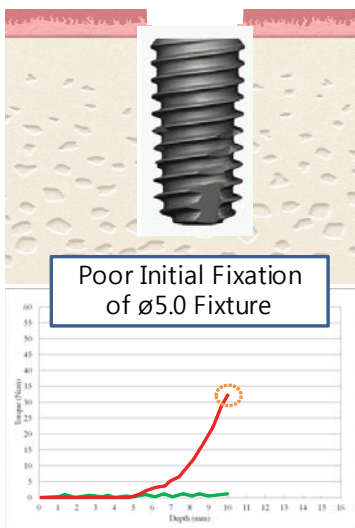

2. Immediate Replacement for Failed $\varnothing 4.5$ Fixture : When $\varnothing 4.5$ Fixture failed and bone quality is weak, $\varnothing 5.5$ Fixture is a good alternative for the position

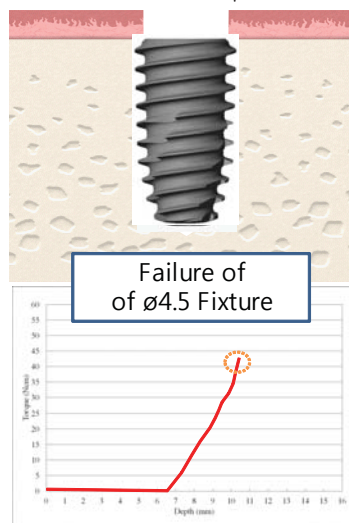

3. Not enough Ridge for Ultra-Wide Fixture : When there is adjacent teeth or not enough ridge for Ultra-Wide fixture, $\varnothing 5.5$ will be selected rather than $\varnothing 6.0$

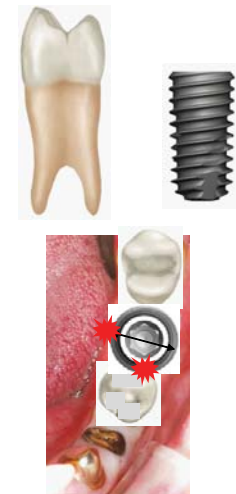


OSSTEM Fixture Line-up

	$\varnothing 3.0$	$\varnothing 3.5$	$\varnothing 4.0$	$\varnothing 4.5$	$\varnothing 5.0$	$\varnothing 5.5$ 5.5 New	$\varnothing 6.0$	$\varnothing 7.0$
TS								
US	-							
SS	-							

Ø5.5 * 10mm




Hard
Bone

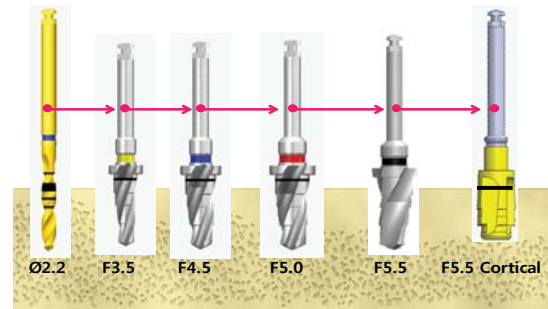


○ → ○ → ○ → ○ → ○ → ○ Drilling 6 times

Normal
Bone

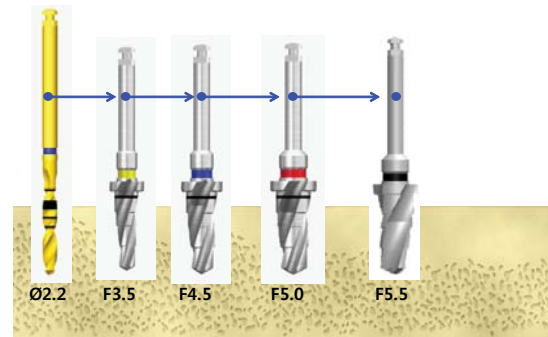

○ → ○ → ○ → ○ → ○ Drilling 5 times

Soft
Bone

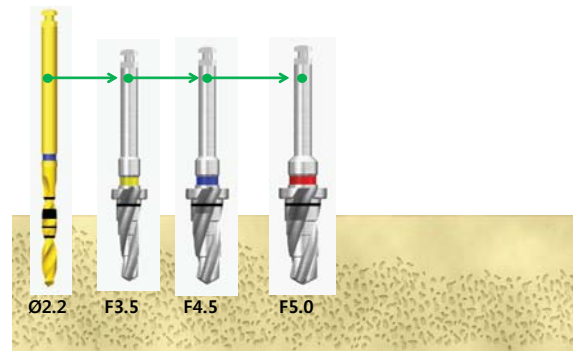

○ → ○ → ○ → ○ Drilling 4 times

OSSTEM[®]
IMPLANT

Ab November 2016
unter www.bedent.ch
erhältlich.